

**Adjustable Shower Stool With Handles  
TRP-90115****Assembly Instructions and Guide**

Thank you for choosing The Ramp People. Please read this guide carefully prior to use.

**Intended Use**

The shower stool is designed to be used as a mobility aid.

**Unpacking**

When unpacking the shower stool please check you have all the required parts and fixings and contact The Ramp People if any items are missing. Please check the item for damage, if any damage is noted please contact the supplier immediately. Do not use a damaged mobility aid.

**Product Specification**

Product Dimensions	
Overall Width	455 mm
Overall Depth	380 mm
Seat Width	380 mm
Seat Depth	280 mm
Seat Height	480 – 630 mm
Width Between Arms	485 mm
Maximum User Weight	160kg (25st)

**Assembly**

To adjust the height, remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has

The Ramp People  
hello@theramppeople.co.uk  
Tel: 01372 569 051

Unit 5 Quantum Business Park, Beacon Hill Road, Fleet GU52 8EA

passed through the extension leg and that it fits neatly and snugly around the outside leg.

### **Cleaning**

Prior to cleaning the stool please remove all pin clips and the four extension legs with the ferrules fitted. Clean these items separately to the main frame.

Clean the plastic seat using a non-abrasive soft cloth with a mild detergent. Abrasive cleaners and/ or abrasive cleaning pads could seriously damage the seat beyond repair.

### **Using the Shower Stool Safely**

- When moving always lift the shower stool by placing your hands on the tubular metal arms. •
- Do not allow children or unauthorised persons to play with the equipment or use without proper supervision.
- Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury.
- Do not use the shower stool outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury.
- Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.

### **Maintenance**

Periodically check to ensure all components are present, correctly assembled and suitably tightened.

### **Warranty**

The supplier provided a 1 year warranty against defects from the original purchase date. Please note the warranty will be void if the mobility aid is used outside of the recommended guidance. No warranty is provided if the shower stool is used for purposes other than as a mobility aid. If you wish to undertake a warranty claim please contact us and provide details of the fault and images via email at [hello@theramppeople.co.uk](mailto:hello@theramppeople.co.uk). The Ramp People are not responsible for incidental or consequential damages.

### **Returns**

The Ramp People offer a 60 day returns period if the product is not suitable for your needs. The item needs to be returned in the original packaging to The Ramp People. Please ensure to package the item correctly as we cannot be held liable for damage caused in transit. Please contact us if you wish to return the item for full returns details.

The Ramp People  
[hello@theramppeople.co.uk](mailto:hello@theramppeople.co.uk)  
Tel: 01372 569 051

Unit 5 Quantum Business Park, Beacon Hill Road, Fleet GU5:

