

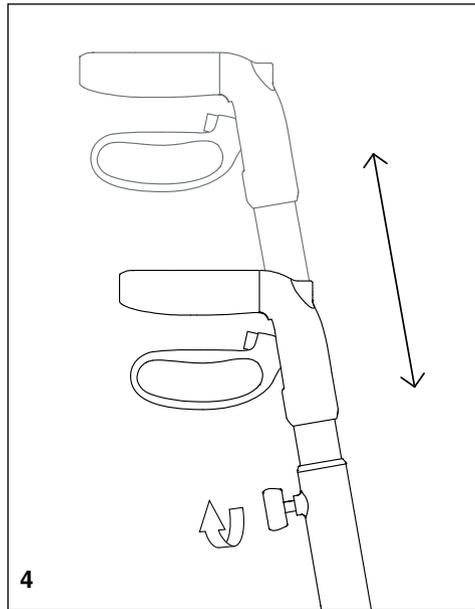
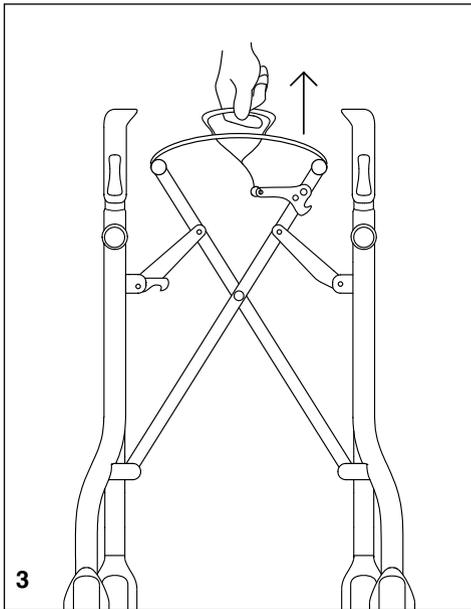


LET'S MOVE

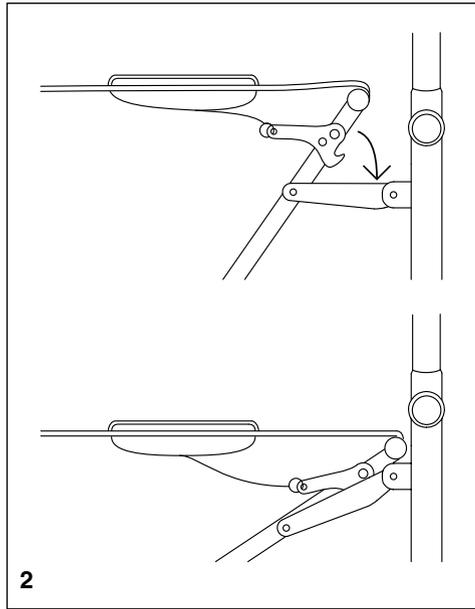

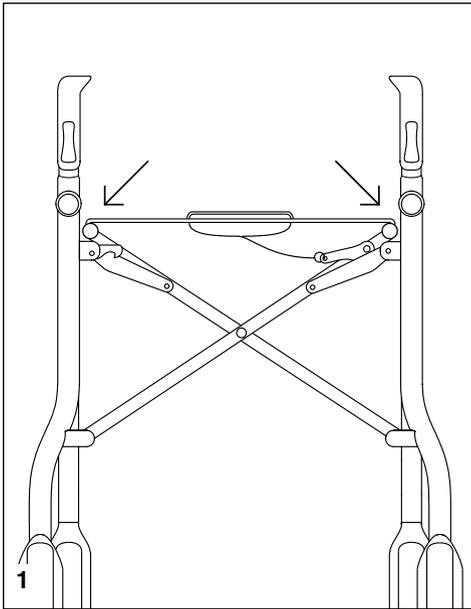

ENGLISH

Let's Move Outdoor Walker.

Congratulations to you for purchasing your new outdoor rollator that will make your life a little easier, more pleasant and more safe.

To make it easier for you to get familiar with your rollator it is important that you study this manual.

The rollator helps you to walk and move outdoors. You can also use it to sit on when you want to rest during a walk. Then do not forget to activate the parking brake – position 3 on the picture to the right.

To cross curbs and other obstacles, walk in a straight angle to the curb for the rollator not to become unsteady as it crosses the edge.

The seat is designed only to sit on, not for transport of for instance children or objects. Do not put heavy objects like large boxes, pieces of furniture etc. on the seat.

The basket is available as an accessory, it can be taken off the rollator and can be used as a shopping bag or alike, also for bringing light objects like fruit, yarn, a thermos, a book and such.

Maintenance: Clean the rollator with not too hot water and a little detergent. Don't put the rollator in hot places or where it is very damp.

Warranty:

We provide a one year warranty. It does not cover wear parts like wires and wheels.

How to unfold.

(See pictures on page 2!)


1. Rollator standing. Unfold the rollator by pushing the two sides apart from each other.
2. Make sure that the rollator is fully unfolded and the locking hook under the seat snaps in place.

How to fold.

3. Lift the seat in the middle and the locking hook under the seat is released and the rollator is folded. Press the sides together if needed.


Handle height adjustment.

4. Loosen the two adjustment screws. Adjust the handle height and tighten screws. Follow the markings on the tube. (Picture 4 page 2!)

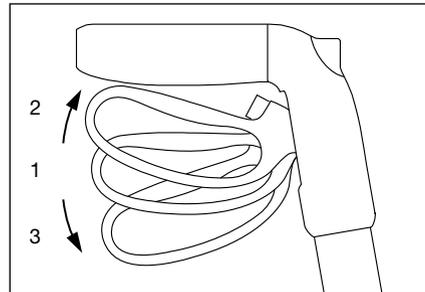

Brake functions.

1. In position 1 the rollator is rolling freely.
2. To brake the rollator pull the handle up to position 2.
3. For parking brake push handle down to position 3, where the brake locks.

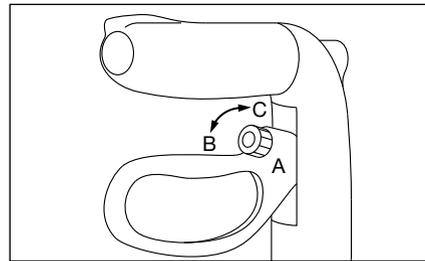

Brake adjustment.

1. The brakes are adjusted with the adjustment screw at the brake handle (A).
2. Adjustment of the wire tension is done by turning the adjustment screw (B-C). If the brake is too loose, turn in direction (B). If the brake is too hard turn in direction (C).

Technical data:




Tested according to ISO 11199-2:2005 for a maximum user weight of 130 kg.

Max. width	58 cm.
Length	65 cm.
Weight	5.4 kg.
Handle height adjustable from	78 – 93 cm.
Seat height	60 cm.
Front wheel diameter	20 cm.
Rear wheel diameter	20 cm.
Max. bag load	12 kg.

MATERIAL

1. Handgrip - TPR
2. Handle Cover - N630
3. Brake Lever - N630
4. Handle Tube - Aluminum
5. Seat Frame - Steel
6. Frame - Aluminum
7. Front Fork - N630
8. Rear Fork - N630
9. Tires - TPR
10. Wheel rim - PP
11. Seat - PVC + Nylon
12. Bushing for handle - PA
13. Folding cross - Steel

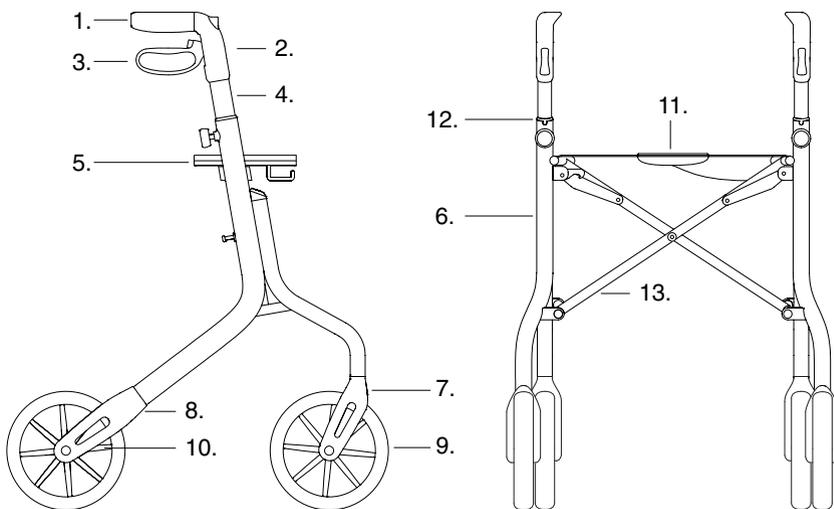

Återvinning / Genbrug / Recycling / Recycling / Recycling / Riciclaggio / Recyclage

Denna produkt skall återvinnas i enlighet med nationella riktlinjer.

Dette produkt skal genbruges i overensstemmelse med nationale retningslinjer.

This product should be recycled in accordance with national guidelines.

Dit product moet worden gerecycled in overeenstemming met de nationale richtlijnen.

Dieses Produkt sollte gemäß den nationalen Richtlinien recycelt werden.

Questo prodotto deve essere riciclato in conformità con le linee guida nazionali.


Ce produit doit être recyclé conformément aux directives nationales.



TRUST CARE

www.trustcare.se

Trust Care
Skrittgatan 8B
21377 Malmö, Sweden
Tel.: +46 40 15 61 20
info@trustcare.se

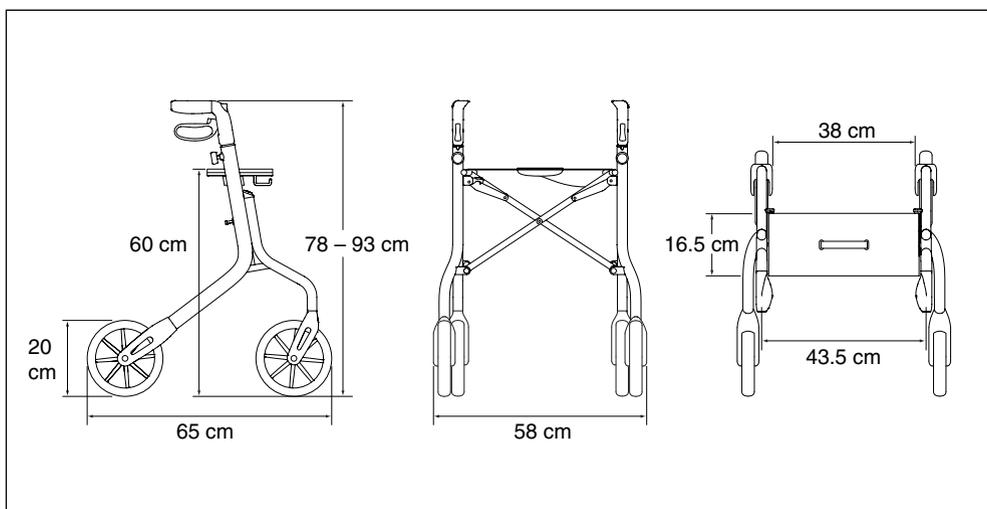

Rev003 Oct 2019