

Adjustable Over Bed Table On Wheels
TRP-90291



Assembly Instructions and Guide

Thank you for choosing The Ramp People. Please read this guide carefully prior to use.

Intended Use

The table is designed to be used as a mobility aid.

Unpacking

When unpacking the table please check you have all the required parts and fixings and contact The Ramp People if any items are missing. Please check the table for damage, if any damage is noted please contact the supplier immediately. Do not use a damaged table.

Product Specification

Width: 410mm

Length: 680mm

Adjustable height: 700mm – 1000mm

Assembly

- 1) Push four castors into the holes in the underside of the frame base.
- 2) Remove the “E Clip” from each leg.
- 3) Place each leg over the base spigots.
- 4) Re-insert the “E Clips” to attach the legs to the spigots.
- 5) Unfold the support legs outwards. One support leg must be re-positioned before proceeding. Press the spring button and slide it under the clip.
- 6) Slacken off the hand wheels and insert the support legs into the legs on the base. Tighten the hand wheels to fix the table top at the required height and angle. Adjusting the angle when the table is in use the hand wheels only require slackening a turn or two they do not need removing.

Cleaning

To clean the table, use a soft and slightly moistened cloth with a mild detergent. It is not recommended to use any bleach, very hot water or any abrasive surface cleaners, as this may result in damage or mark the frame surface.

The Ramp People
hello@theramppeople.co.uk
Tel: 01372 569 051
Unit 5 Quantum Business Park, Beacon Hill Road, Fleet GU52 8EA

Operation Instructions

To adjust the height and/or angle of the table, first slacken the 2 short hand screws by turning anticlockwise. Adjust the table to suit, and secure by turning hand screws clockwise. The table is not load bearing, and should not be used to support objects over 5kg (11lb). Please do not allow people to lean on the table.

Maintenance

Periodically check to ensure all components are present, correctly assembled and suitably tightened.

Warranty

The supplier provided a 1 year warranty against defects from the original purchase date. Please note the warranty will be void if the table is used outside of the recommended guidance. No warranty is provided if the table is used for purposes other than as a mobility aid. If you wish to undertake a warranty claim please contact us and provide details of the fault and images via email at hello@theramppeople.co.uk. The Ramp People are not responsible for incidental or consequential damages.

Returns

The Ramp People offer a 60 day returns period if the product is not suitable for your needs. The item needs to be returned in the original packaging to The Ramp People. Please ensure to package the table correctly as we cannot be held liable for damage caused in transit. Please contact us if you wish to return the table for full returns details.

The Ramp People

hello@theramppeople.co.uk

Tel: 01372 569 051

Unit 5 Quantum Business Park, Beacon Hill Road, Fleet GU52 8EA